

Educational Resource



Health
Hunter New England
Local Health District

Wound Management: Prontosan Solution

Sites where the resource can be utilised: All clinical sites where Prontosan Solution is used

Target audience: Clinicians

Purpose: Education related to Prontosan Solution product, application and use guide

Description: Educational information relating to Prontosan Solution

Learning Outcomes: Identify indication/contraindications of Prontosan Solution use
Describe how to apply Prontosan Solution

Keywords: Prontosan Solution, PHMB, cleansing solution, dressing

Resource registration number

Replaces existing resource? No

Related Legislation, Australian Standard, NSW Ministry of Health Policy Directive or Guideline, National Safety and Quality Health Service Standard (NSQHSS) and/or other, HNE Health Document, Professional Guideline, Code of Practice or Ethics:

NSQHS Standards Criterion and/or other: 1, 3, 8

- [NSW Health Policy Directive Clinical Procedure Safety PD 2014_036](#)

http://www0.health.nsw.gov.au/policies/pd/2014/pdf/PD2014_036.pdf

- [NSW Ministry of Health Policy PD 2005_406 Consent to Medical Treatment](#)

http://www.health.nsw.gov.au/policies/PD/2005/pdf/PD2005_406.pdf

- [NSW Ministry of Health Policy Directive PD 2007_036 Infection Control Policy](#)

http://www.health.nsw.gov.au/policies/pd/2007/pdf/PD2007_036.pdf

- Australian Wound Management Association Standards for Wound Management ST3

http://www.awma.com.au/publications/2011_standards_for_wound_management_v2.pdf

Is this package recorded in HETI?

No

Contact person: Jayne Campbell


Contact details: jayne.campbell@hnehealth.nsw.gov.au

Date accepted by LP committee: N/A

Issue date: August 2015

Review date: August 2018

To view the PowerPoint presentation, double click on the image below and scroll down.



Only a Clean Wound Can Heal

Experience the Prontosan® effect.

BAUS OM E0039 05/15


B | BRAUN
SHARING EXPERTISE

Double click on image above to view presentation.

To print presentation

1. Right click on image, select “presentation object”, and then “open”.
2. The presentation will open in a format that is able to be printed.

Acticoat Absorbant

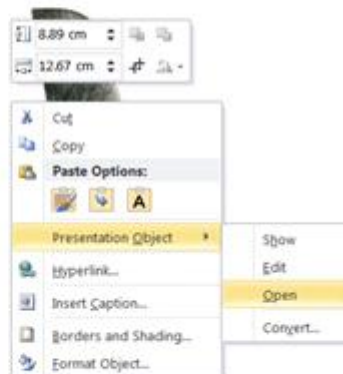

Double click on image to view presentation.