

## Educational Resource



**Health**  
Hunter New England  
Local Health District

### Wound Management: Mepitel

Relevant for use at clinical sites where the product is used

**Target:** Clinicians      **My Health Learning:** N/A

**Purpose:** Information/ demonstration of application and use of the product

**Description:** This resource provides information/ demonstration on the use of the product using a presentation. These can be used a part of a broader training program for wound management or as a standalone resource to inform the use of this specific product.

**Learning outcomes:** Identify indication/contraindications of Mepitel  
Describe how to apply Mepitel

**Keywords:** Mepitel , scar management, safetac, contact layer, silicone, dressing

**Related Legislation, Australian Standard, NSW Ministry of Health Policy Directive or Guideline, National Safety and Quality Health Service Standard (NSQHSS) and/or other, HNE Health Document, Professional Guideline, Code of Practice or Ethics:**

NSQHS Standards Criterion and/or other: 1, 3, 8

[NSW Health Policy Directive Clinical Procedure Safety PD 2017\\_032](#)

[NSW Ministry of Health Policy PD 2005\\_406 Consent to Medical Treatment](#)

[NSW Ministry of Health Policy Directive PD 2017\\_032 Infection Control Policy](#)

[Wounds Australia 2016 Standards for Wound Management Prevention and Management 3<sup>rd</sup> Ed](#)

**Contact:** Jayne Campbell [jayne.campbell@hnehealth.nsw.gov.au](mailto:jayne.campbell@hnehealth.nsw.gov.au)


**Issued:** August 2015 **Review:** August 2018

### How To Use Mepitel

*[Double click to view the presentation](#)*

**Mepitel®**

Wound contact layer with Safetac® technology



- **Product Description**
- Mepitel consists of:
  1. a Safetac® wound contact layer on BOTH sides
  2. a porous, transparent and flexible polyamide net with open mesh structure

MÖLNLYCKE HEALTH CARE

Safetac® TECHNOLOGY